



Fitting a Respirator Mask

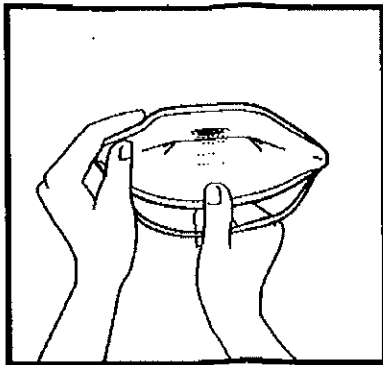


DIRECTIONS FOR APPLICATION


▲1. Separate the edges of the respirator to fully open it.

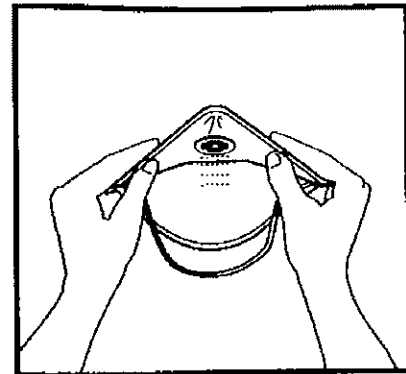


▲2. Slightly bend the nose wire to form a gentle curve.

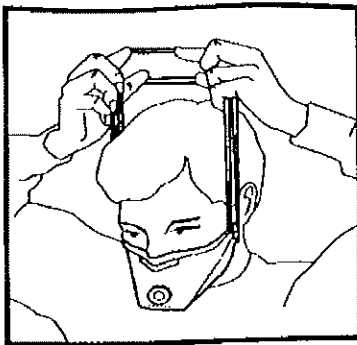


▲3. Hold mask in position over nose and mouth, with retaining straps around back of head. Pull lower portion of strap over head and fit around neck.

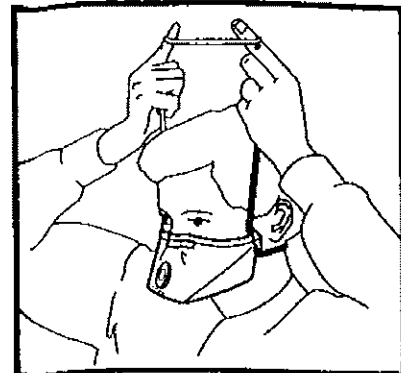

▲4. Stretch and pull the upper strap over your head on the back of your head above your ears.

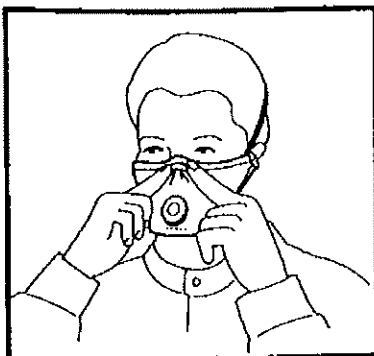

▲5. Using both hands, form nose clip around nose and adjust the tension in the upper and lower portions of the continuous loop head strap, to ensure a positive face seal.

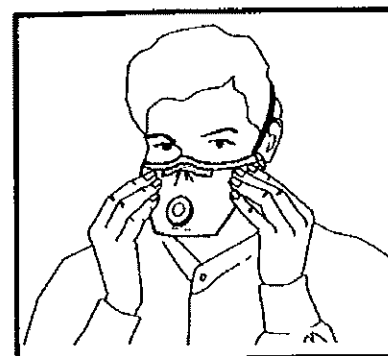

▲6. To test fit for respirator with exhalation valve: Cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the headband for better fit.